

Contraindications

Treatment Type	Contraindication	Category	Reasoning
All treatments	Severe Raynaud's Syndrome	Absolute NO	Due to the application of cold temperatures inherent to every treatment.
All treatments	Severe allergy to cold	Absolute NO	Due to the application of cold temperatures inherent to every treatment.
All treatments	Cold-related illness	Absolute NO	Due to the application of cold temperatures inherent to every treatment.
All treatments	Lower Limb Ischemia	Absolute NO	Lower Limb Ischemia is a severe blockage in the arteries of the lower extremities, which markedly reduces blood flow. The application of heat and cold has an impact on localized circulation and the vascular system.
All treatments	Fillers in the treatment area in the past 90 days	Absolute NO	Treatment prior to the end of the 90-day waiting period may be disruptive to the filler injections and may actually move the fillers beneath the skin.
All treatments	Bacterial and viral infections of the skin	Absolute NO	Risk of transmission and increased safety risk to the client.
All treatments	Active/Severe Eczema, Rashes, or Dermatitis in the desired treatment area	Absolute NO	Due to the risk of spreading or worsening these skin issues and producing additional irritation.
All treatments	Silicone/other implants in the desired treatment area	Absolute NO	Due to the application of heat and subzero temperatures directly to the skin surface, there is the risk of disruption to implants.
All treatments	Irremovable body piercings in the desired treatment area	Absolute NO	Metal will damage the wand head and there are also concerns around client safety.

All treatments	Open or infected wounds in the desired treatment area	Absolute NO	Risk of infection.
All treatments	Impaired skin sensation in the desired treatment area	Absolute NO	Due to the application of cold temperatures inherent to every treatment.
All treatments	Known sensitivity or allergy to propylene glycol	Absolute NO	Propylene Glycol is an essential ingredient in the Cryoskin gel. ONLY the Cryoskin gel may be used when applying treatments as it has been specially formulated to reach the peak heat and cold temperatures without crystallizing or burning on the clients' skin.
All treatments	Past Cancer	Consult physician	Depending on the type of cancer and the impacts of cancer treatment the client may not be eligible for treatment.
All treatments	HIV/AIDS	Consult physician	Due to the impact of various bodily systems inherent to these illnesses.
All treatments	Progressive Diseases (ex: MS, ALS, Parkinson's, Neuropathy)	Consult physician	The application of intense heat or cold inherent to the treatments may lead to a more rapid degeneration of the nerve endings which are progressively degenerating leading to extended numbness and other issues.
All treatments	Cardiovascular Disease	Consult physician	The application of heat and cold has an impact on localized circulation and the vascular system.
All treatments	Wound healing disorders	Consult physician	Client safety concern - inflammatory and scarring.
All treatments	Circulatory disorders	Consult physician	The application of heat and cold has an impact on localized circulation and the vascular system.
All treatments	Use of topical antibiotics in the desired treatment area	Consult physician	Disruption of the antibiotics and increased risk of skin irritation.

All treatments	Metal implants in or adjacent to the desired treatment area	Consult physician	Due to the application of heat and subzero temperatures directly to the skin surface, there is the risk of disruption to those implants.
All treatments	Mesh inserts in or adjacent to the desired treatment area	Consult physician	Potential disruption of the insert.
All treatments	Hernia in or adjacent to the desired treatment area	Consult physician	Possibility of disruption of the healing process or tissue.
All treatments	An active implanted device such as a pacemaker or defibrillator	Consult physician	Unknown effects on functional implants.
All treatments	Active/Severe Eczema, rashes, or dermatitis outside of the desired treatment area	Consult physician	If the client is having a current outbreak they may be more prone to skin irritations and sensitivity.
All treatments	Any serious health condition not specified	Consult physician	If your client presents with a serious health concern not listed, please ask that they consult with their physician before receiving treatments.
CryoSlimming & CryoToning	Pregnancy/ Breastfeeding	Absolute NO	The effects of accelerated metabolism on the fetus and/or breastmilk are unknown.
CryoSlimming	Active Cancer	Absolute NO	Consult with a physician. Depending on the type of cancer and the impacts of cancer treatment the client may not be eligible for treatment.
CryoSlimming	Uncontrolled Diabetes or Diabetes-Related Complications	Absolute NO	Due to impact on the lymphatic system with the CryoSlimming.
CryoSlimming	Severe Kidney or	Absolute NO	Due to impact on the lymphatic

	Liver Disease		system with the CryoSlimming.
CryoSlimming	Lymphatic Disorders	Consult physician	Due to impact on the lymphatic system with the CryoSlimming.
CryoSlimming	Any surgery in the past six months	Consult physician	Depending on the surgery site and type of surgery treatment might interfere with the healing process due to the impact on the lymphatic system.
CryoToning & CryoFacial	Surgery in or adjacent to the desired treatment area in the past 6 months	Consult physician	Depending on the surgery site and type of surgery treatment might interfere with the healing process.
CryoToning & CryoFacial	Active Cancer	Consult physician	Depending on the type of cancer and the impacts of cancer treatment the client may not be eligible for treatment.
CryoFacial	Botox in the past 30 days	Absolute NO	Treatment prior to the end of the 30 day waiting period may be disruptive to the Botox injections.
CryoFacial	PDO Threads in the past 90 days	Absolute NO	Treatment prior to the end of the 90 day waiting period may be disruptive to the PDO threads.